

# GREEN VAC

## *facial*

- A unique combination of facial vacuum massage and diamond microdermabrasion for healthy and youthful skin
- Non-invasive and non-wounding treatments that entail no discomfort, down time or recovery
- Effective on all skin colors and types
- Cost-effective procedures requiring no consumables
- A variety of applicators for a wide range of applications
- Preset programs as well as the programs free to be set by the user



CE 0123



The **Green VAC Facial** is a cost-effective solution allowing the practitioner to perform two facial care procedures: **vacuum massage** (so called **radial endodermotherapy**) and **diamond microdermabrasion**.



Facial radial endodermotherapy is based on the effects of intermittent vacuum. It significantly increases capillary permeability what accelerates the supply of natural nourishing substances, oxygen, oligo-elements, and enzymes. Additionally, it improves the hydration of extracellular matrix what leads to the stimulation of fibroblasts activity followed by the activation of new cell growth. As a consequence, the collagen and elastic fibers are remodeled, and the tone of facial muscles is improved what reduces the appearance of wrinkles and creases.



The use of deep tissue vacuum massage is not limited to the facial regions only. The **Green VAC Facial** has been proven to be an excellent tool for neck rejuvenation as well as for the treatment of upper arms.

#### Technical data:

Model:	<b>GREEN VAC Facial</b>
Code:	<b>1800357</b>
Vacuum:	0 to 500 HPa, adjustable
Duration of Vacuum High and Low:	0 to 60 seconds
Therapy time	0 to 99 minutes
No. of channels:	1
No. of fixed programs:	5
No. of free programs:	5
Weight:	3 kg
Dimensions:	300 x 300 x 140 mm
Voltage:	230 V, 50/60 Hz
Power:	50 VA
Fuse:	2 x 250 mA

Classification according to IEC 60601/1: Class I, Type BF  
 Classification according to MDD 93/42/EEC: Class IIa

#### Basic set:

Set of facial applicators AF	1 pc
Applicator VPG60	1 pc
Applicator VPG35	1 pc
Applicator VPG25	1 pc
Applicator RAPG35	1 pc
Connector with filter CK	1 pc
Sponge filter	10 pcs
Mains cable (230 V, EURO plug)	1 pc
Operators Manual	


#### Option:

Set of facial applicators VG	
Diamond set (code 1512095)	
Diamond set MAX (code 1512112)	
Diamond set AQUA (code 1512111)	

#### Set of facial applicators AF



#### Applicators

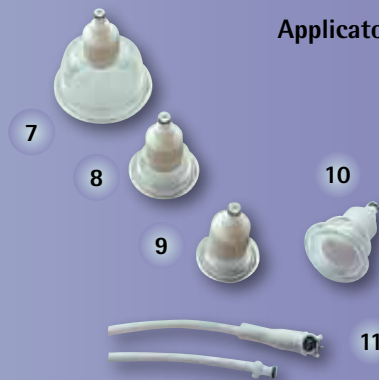

**Diamond set** with 50, 75 and 100 microns tips for precise microdermabrasion procedure!



1. **1512 060** Facial applicator  $\varnothing$ 25 AF25
2. **1512 061** Facial applicator  $\varnothing$ 20 AF20
3. **1512 062** Facial applicator  $\varnothing$ 15 AF15
4. **1512 063** Facial applicator  $\varnothing$ 11 AF11
5. **1512 064** Facial applicator  $\varnothing$ 8 AF8
6. **1512 065** Holder for facial applicator AAF
7. **1512 040** Applicator  $\varnothing$ 60 VPG60
8. **1512 041** Applicator  $\varnothing$ 35 VPG35
9. **1512 042** Applicator  $\varnothing$ 25 VPG25
10. **1512 012** Radial applicator  $\varnothing$ 35 RAPG35
11. **1512 080** Connecting tube with filter CK

Iskra Medical d.o.o

Stegne 23, 1000 Ljubljana, Slovenia, phone: ++386 (0)1 513 15 00, 513 15 04, fax: ++386 (0)1 513 15 10, mobile: ++386 41 622 918, E-mail: medical@iskramedical.eu, http://www.iskramedical.eu

BEAUTY TEC LTD

62 Church Road · Barnes · London SW13 0DQ · T: 020 3184 0001 E: info@beauty-tec.co.uk · www.beauty-tec.co.uk

